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
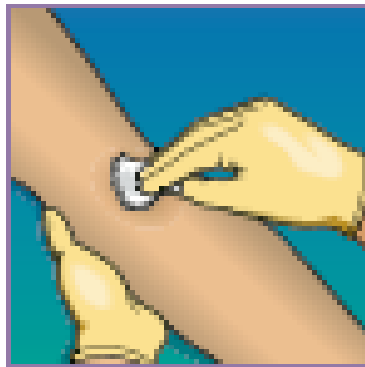
STEP 1. SKIN PREPARATION

- Cleanse the venipuncture site with 70% isopropyl alcohol.
- Starting at the middle of the site, swab concentrically with a 1 to 10% povidone-iodine solution or chlorhexidine-gluconate.

NOTE: Chlorhexidine-gluconate is recommended for infants two months and older and patients with iodine sensitivity.


- Allow the site to air dry.

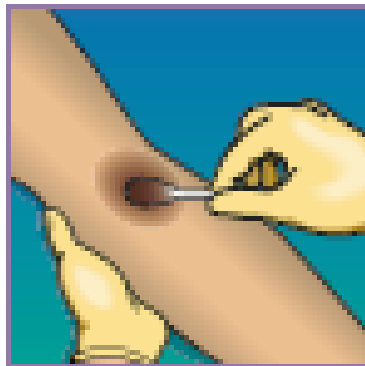NOTE: If the venipuncture proves difficult and the vein must be touched again to draw blood, the site should be cleaned again.

STEP 2. PREPARE BACTEC™ VIALS

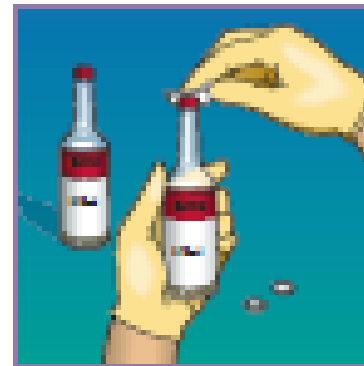- Mark BACTEC culture vial label(s) at desired fill level.
- Remove flip-off caps from BACTEC culture vial(s).
- Wipe tops of vials with single alcohol swab and allow to dry.

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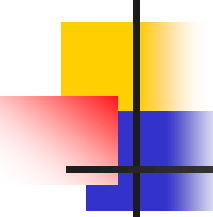
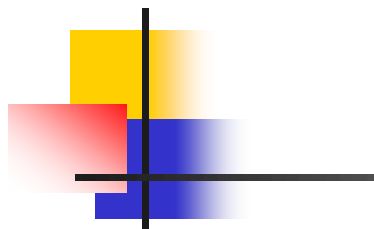
min. 3-8 ml

CAUTION:
reduced
sensitivity

- Optimal blood fill volumes critically determine the diagnostic yield of your blood cultures. Each ml of blood, up to 10 ml, can increase the sensitivity of the blood culture by 3–5%^{5,6}. While overfilling of bottles may cause false-positivity, underfilling reduces the sensitivity of blood cultures.
- Laboratories should routinely monitor the volume of blood cultured as a quality assurance activity and provide feedback to clinical staff to insure an adequate volume of blood is drawn for blood culturing.
- However, manual blood volume monitoring is time consuming and tedious, therefore not commonly done.

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DO NOT COVER BOTTLE BARCODE.

On the bottle, write tech # or initials of person drawing, time of collection, and site (line, red port, etc) if applicable.

DO NOT WRITE OVER THE BARCODES.

BD EpiCenter™ - improving efficiency and patient care via real-time communication.

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